

schülke wet wipes dispensing system

Filling under hygienic conditions

Step 1

For personnel protection as well as protection of the schülke wipes from contamination, we recommend prior disinfection of the work surface with for example terralin® liquid or mikrozid® AF liquid and wear protective coats and gloves.



Step 2

Open the lid of the wet wipe dispenser and insert the unpacked fleece roll into the dispenser. Pull the inmost wipe a little from the centre.



Step 3

Then slowly pour 2 – 2.5 litres of ready for use disinfectant solution over the fleece roll, spirally from the centre to the outside. This ensures complete wetting.



Step 4

Now pull the inmost wipe of the roll into the star-shaped dispensing opening of the lid and dispose of it.



Step 5

Place the lid onto the dispenser and close it until it locks into place. Now carefully close the protective cap of the dispensing system.



Step 6

After a minimum of 20 minutes, the wipes are completely saturated.



Wait
20 minutes



Step 7

In the meantime, complete each charge label supplied with each cloth roll and stick it onto the field provided.



Step 8

The schülke wet wipes dispensing system is now ready for use.



Discard solution



Dispose of wipes



Clean & dry



Spray down

Refilling

Please dispose of any remaining products and wipes after the expiry date or when refilling accordingly. Then thoroughly clean and disinfect the dispenser. If required, it can be thermally recycled up to 70 °C. The dried dispenser is refilled in the same way as for a first filling. We recommend to additionally spray down the lid with e.g. terralin® liquid or mikrozid® AF liquid. Please always observe hygienic requirements for producing and storing disinfectant solutions.