

schülke -†



The cleaning intensifier for gigasept® FF is an alkaline cleaner. It contains a special corrosion protection for aluminium.



Cleaning booster for gigasept® FF

Our plus:

- very gentle to materials
- very good cleaning power regarding blood, serum and proteins

Areas of application

The cleaning intensifier for gigasept® FF is applied during reprocessing of flexible endoscopes for pretreatment or mixed with gigasept® FF.

Application methods / Instructions for use

1 % cleaning intensifier for gigasept® FF is always added to the already prepared stock solution of gigasept® FF with a separate measuring cup. Avoid contact between the two concentrates.

This solution needs to be renewed every day.

It should be avoided that the concentrate and the stock solution of the cleaning intensifier for gigasept® FF dry on surfaces or on instruments, as insoluble residues may otherwise develop.

Compatibility of materials

- not suitable for alkali-sensitive materials, e.g. zinc
- suitable for aluminium

Application concentrations / Contact times

1 % (i.e. 10 ml for 1 litre of water) to be added to the respective prepared stock solution of gigasept® FF.

If mixed with gigasept® FF, the contact time depends on the contact time specified for the disinfectant.

Product data

Labelling according to Regulation (EC) 648/2004:
< 5 % phosphonate

Further ingredients:
corrosion inhibitors, pH regulators

Chemical / Physical data:

Concentrate:

Appearance:	clear, yellowish liquid
pH value:	approx. 12.5

Labelling according to EC directives

Xi: Irritant.

R36/38: Irritating to eyes and skin.

S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Cleaning booster for **gigasept® FF**

Accessories / Dosing aids

	Art. No.
s&m bottle dosing device for the 2 l bottle (20 ml/1 x tilting)	183 401
Measuring cup 50 ml	136 102
Decanting hopper	117 901
instrument tubs 3, 5, 10, 30 l	

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinions and information

Should you have any further questions:

Customer Care

Phone: +49 (0)40 521 00-666

E-mail: info@schuelke.com



Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).

Schülke & Mayr GmbH
22840 Norderstedt, Germany
Phone +49 (0) 40 - 521 00 - 0
Fax +49 (0) 40 - 521 00 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1AT
Phone +44 (0) 1142 - 54 35 - 00
Fax +44 (0) 1142 - 54 35 - 01
mail.uk@schuelke.com

Schülke & Mayr Ges.m.b.H
Seidengasse 9
1070 Wien, Austria
Phone +43 (0) 1 - 523 25 01 - 0
Fax +43 (0) 1 - 523 25 01 - 60
office.austria@schuelke.com