



Ready to use disinfectant wipes
impregnated with an alcoholic solution.

Now also in
softpacks!

mikroloid® AF wipes

Our plus:

- ready to use, impregnated disinfection wipes
- very broad biocidal activity within the shortest possible time
- dries rapidly
- now also in „premium quality“ in a softpack!

Areas of application

Alcohol-based rapid disinfection of medical devices [CE 0297] in all areas with an increased risk of infection and where short contact times are required, e.g.:

- Patient treatment units
- Examination couches
- Operating tables and nearby work surfaces
- Surfaces of medical equipment

Instructions for use

Wipe areas thoroughly with the impregnated cloth; Ensure thorough wetting and leave to work. If necessary, use several wipes for large areas.

Microbiological effectiveness

- Bacteria (including TB) • Fungi • Hepatitis B viruses • HIV • HCV
- Adenoviruses • Rotaviruses • Papovaviruses • Polioviruses

Contact times

Effective against	Contact times
Bacteria, fungi (<i>C. albicans</i>) (in accordance with DGHM Guidelines with short contact times)	1 min
MRSA Multiresistant <i>Staphylococcus aureus</i> strains	1 min
TB	1 min
Fully virucidal*	30 min
Effective against enveloped viruses* (incl. HIV, HBV, HCV)	30 sec
Adenoviruses (type 2)	2 min
Noroviruses**	1 min
Rotaviruses	30 sec
Polioviruses	30 min
Polyoma SV 40	15 min
Vaccina viruses	30 sec
DGHM list / certificate Surface disinfection in hospitals and medical practices	5 min
ÖGHMP Expertise No. 95/8	1 min

* In accordance with the Robert Koch Institute guideline – German Federal Health Gazette 01/2004

** Murine Noroviruses according to EN 14 476

mikrozyd® AF wipes

Product data

Composition

100g agent solution contain the following active ingredients:
25 g Ethanol (94 %), 35 g Propan-1-ol

Chemical / physical data:

Active solution:

Appearance of solution:	clear, colourless liquid
Density (20 °C):	approx. 0.89 g/cm ³
Thermostability:	-5 °C to +40 °C
pH:	approx. 6.0
Flash point:	27 °C (DIN 51 755)

Delivery form / unit

Size of trading unit	Unit	Art. No.
mikrozyd® AF wipes 150 wipes-dispenser 150 wipes-refill	10 tins	109 123
	20 refills	109 124
mikrozyd® AF wipes premium 24 wipes softpack 48 wipes softpack	14 packs	119 103
	18 packs	119 104

Labelling according to EC Directives

R10: Flammable.

S2: Keep out of the reach of children

Special instructions

Avoid contact with the eyes.

The wearing of protective gloves (e.g. of nitril rubber) is recommended. Do not use on alcohol-sensitive surfaces (e.g. acrylic glass).

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinions and information

Please visit our website for an overview of all available literature/ reports on mikrozyd® AF wipes: www.schuelke.com

For your individual inquiries:
Customer care

Phone: +49 (0)40-521 00-666

Email: info@schuelke.com

The advantages of the mikrozyd® AF wipes premium

- handy softpack format
- premium quality wipe
- extra soft
- high coverage



BAUA: N-40580 (PT2)



Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).

Schülke & Mayr GmbH
22840 Norderstedt, Germany
Phone +49 (0) 40 - 521 00 - 0
Fax +49 (0) 40 - 521 00 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1AT
Phone +44 (0) 1142 - 54 35 - 00
Fax +44 (0) 1142 - 54 35 - 01
mail.uk@schuelke.com

Schülke & Mayr Ges.m.b.H
Seidengasse 9
1070 Wien, Austria
Phone +43 (0) 1 - 523 25 01 - 0
Fax +43 (0) 1 - 523 25 01 - 60
office.austria@schuelke.com