

schülke -†

octenilin® wound irrigation solution

The fast and effective solution for wound cleansing.



the plus of pure performance



octenilin® wound irrigation solution
cleans wounds fast and effectively.

Fields of application

- fast and effective wound cleansing
- removes wound coating and biofilm
- moistens wounds and creates an ideal environment for wound healing
- for preserving moisture of dressings and wound coverings
- loosens encrusted dressings gently and painlessly

Application spectrum

- application from the bottle
- rubbing using a soaked swab
- soaking a compress left on the wound to loosen biofilm and wound coating

Application advantages

- excellent cleaning performance
- especially well tolerated by the skin and tissues
- painless application
- for repeated, long-term use
- suitable for all wound situations
- best before 8 weeks after the first use

Ingredients with many advantages


Octenidine prevents bacteria and fungi from growing in the solution and the wound dressing.

Ethylhexyl glycerin reduces the surface tension and ensures excellent cleaning performance.

octenilin® wound irrigation solution loosens the biofilm by means of its strong cleansing power.

Excellent moisturising property


The ingredient ethylhexyl glycerin reduces the surface tension and thus ensures optimal moistening of the skin. Even wound sections which are difficult to access can be optimally moistened and cleaned.

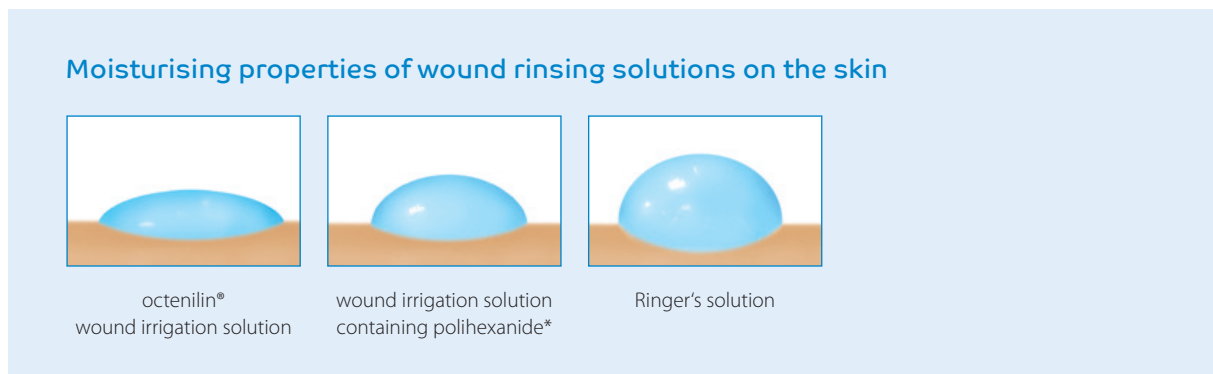

Best cleansing power against biofilms

The good cleansing power of octenilin® wound irrigation solution loosens the biofilm and thus the microorganisms contained in it. Through intensive rinsing with octenilin® wound irrigation solution, biofilm and microorganisms are washed out of the wound, which results in a significant germ reduction in the wound.

Inactivation of biofilm: comparison of different wound irrigation solutions (after 5 and 60 min. contact time)

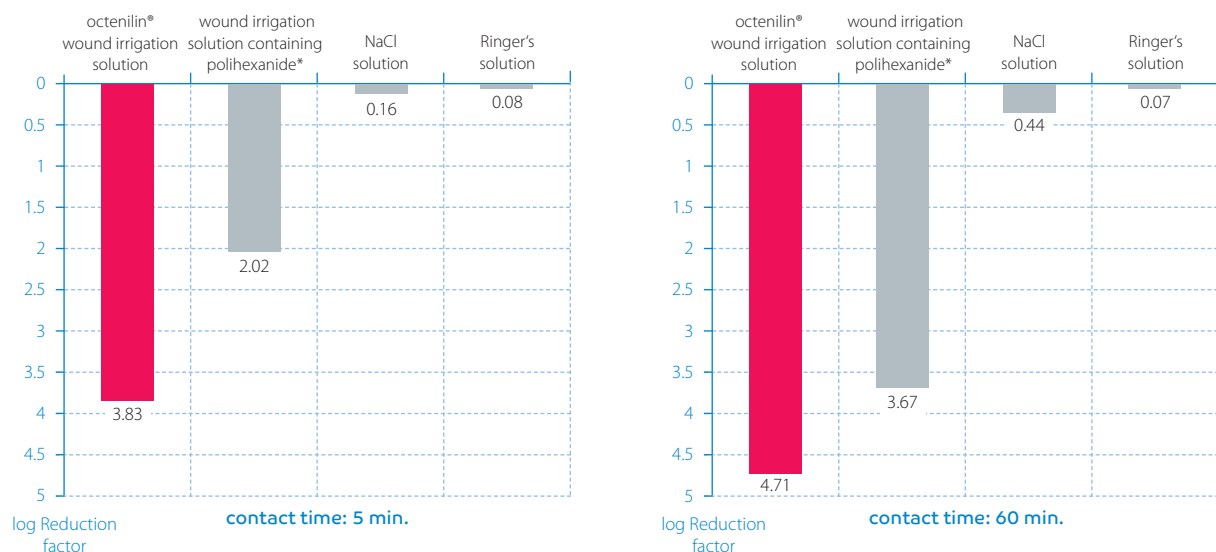

* 0.02 % polihexanide



octenilin[®], the wound irrigation solution with the best cleansing power.



octenilin[®] wound irrigation solution

For fast, effective and gentle wound irrigation.

Our Plus

- excellent cleaning performance
- especially well tolerated by the skin and tissues
- painless application
- for repeated, long-term use
- best before 8 weeks after the first use

Package size

350 ml bottle

Art-No.

121 701

Product data

Ingredients: Aqua valde purificata, Glycerin, Ethylhexylglycerin, 0.05 % Octenidine HCL

Special advice

In order to prevent possible tissue damage, care must be taken that the preparation is not entered or injected under pressure into the tissue. In case of wound cavities, make sure that there is always a drain (e.g. drainage, strap). Do not swallow octenilin[®] in larger quantities, neither let it enter into the blood circulation, e.g. by means of an inadvertent injection.

Schülke & Mayr GmbH
 22840 Norderstedt | Germany
 Phone | Fax +49 40 521 00-0 | -318
 www.schuelke.com
 A company of the
 Air Liquide Group

