



Liquid concentrate based on a combination of quaternary ammonium compounds, aromatic alcohols, amphoteric glycine derivatives and non-ionic surfactants for disinfecting and cleaning surfaces

## perform® classic concentrate Q-Plus

### At a glance:

- Fast and effective
- Excellent material compatibility
- Efficacy tested to European norms

### Applications

Disinfection and cleaning of all types of surfaces in all areas where a guarantee of hygiene is required.

### Instructions for use

Prepare active solution in desired concentration. schülke dispensing aids help ensure exact dispensing of the preparation, dispensing is possible via centralised and distributed dispensing devices. Wipe surfaces, floors and other wipeable areas with a moist cloth, ensuring that all areas are moistened completely. perform® classic concentrate Q-Plus has excellent cleaning properties, therefore the use of additional cleaning additives is unnecessary. Depending on the type and quality of the floor, supplementary floor care may be necessary. Do not combine this product with other products without first consulting schülke, this may lead to reactions with anionic surfactants due to the content of cationic active ingredients.

### Microbiological effectiveness

perform® classic concentrate Q-Plus is: bactericidal, virucidal (BVDV, Vaccinia), fungicidal.

### Contact times / Usage concentration

bactericidal	0,5 % (5 ml/l)	60 Min.
fungicidal	0,5 % (5 ml/l)	60 Min.
virucidal (BVDV, Vaccinia)	0,5 % (5 ml/l)	60 Min.



Tested according to EN 1276, 1650, 13697.

### Product data

Composition: 100 g contain: 22 g Benzalkonium chloride (benzyl-C12-18-alkyldimethylammonium, chlorides), 17 g 2-phenoxyethanol, 0,9 g amino alkyl glycine (Amines, n-C10-16-alkyltrimethylenedi-, reaction products with chloroacetic acid).

Labelling according to Regulation (EC) No. 648/2004  
5 – 15 % non-ionic surfactants, perfumes.

### Chemical / physical data:

Appearance:	clear, green liquid
Density at 20 °C:	1,01 g/cm <sup>3</sup>
Odour:	pleasant
pH Concentrate:	ca. 8,6
pH 0,5 % solution:	ca. 8 – 9

# perform<sup>®</sup> classic concentrate Q-Plus

## Labelling according to EC Directives

- C: Corrosive.  
R22: Harmful if swallowed.  
R34: Causes burns.  
S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.  
S35: This material and its container must be disposed of in a safe way.  
S36/37/39: Wear suitable protective clothing, gloves (e.g. made of butyl rubber) and eye/face protection.  
S45: In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

## Special notes

Use disinfectants safely. Always read the label and product information before use.

## Forms of delivery/packaging units

Container sizes	Packaging units	Art. No.
2 L bottle	5 x 2 L	181622
5 L canister	1 x 5 L	181625

## Accessories/dispensing aids

	Art. No.
2 L bottle dispenser (20 ml/tip 1 time)	183401
5 L canister outlet tap	135501
5 L canister dispenser (20 ml/time)	117101
5 L canister spanner	135810
500-ml measuring cup	136101
Decanting funnel	117901

For detailed information on our dispensing devices, please request the complete overview of hygiene technology.

## Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

## Expert opinions and information

Please visit our website for an overview of all available literature / reports on perform<sup>®</sup> classic concentrate Q-Plus: [www.schuelke.com](http://www.schuelke.com)

For further information please contact your local representative or local offices.



Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).

Schülke & Mayr GmbH  
22840 Norderstedt, Germany  
Phone +49 (0) 40 - 521 00 - 0  
Fax +49 (0) 40 - 521 00 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[mail@schuelke.com](mailto:mail@schuelke.com)

Schülke & Mayr UK Ltd.  
1 Jenkin Road  
GB-Sheffield S9 1AT  
Phone +44 (0) 1142 - 54 35 - 00  
Fax +44 (0) 1142 - 54 35 - 01  
[mail.uk@schuelke.com](mailto:mail.uk@schuelke.com)

Schülke & Mayr Ges.m.b.H  
Seidengasse 9  
1070 Wien, Austria  
Phone +43 (0) 1 - 523 25 01 - 0  
Fax +43 (0) 1 - 523 25 01 - 60  
[office@schuelke.at](mailto:office@schuelke.at)