

schülke -t



Ready-to-use impregnated wipes for disinfecting surfaces

perform® classic wipes EP

At a glance:

- Fast and effective
- Economical with 200 jumbo wipes (approx DIN A4 size) per can
- Efficacy tested to European norms

Applications

For disinfection of surfaces in areas where fast microbiological effectiveness is required.

Instructions for use

Tear off wipes downward at an angle. Thoroughly wipe surfaces with the impregnated wipe, ensure that all areas are moistened completely, and allow the solution to work. Close the lid tightly after use. Do not use on surfaces that are particularly sensitive to alcohol, such as acrylic glass.

Labelling according to EC Directives

R10: Flammable.

Microbiological effectiveness

perform® classic wipes EP are: bactericidal, fungicidal, virucidal (BVDV, Vaccinia).

Contact times

bactericidal	1 Min.
fungicidal	1 Min.
virucidal (BVDV, Vaccinia)	1 Min.



Tested according to EN 1276, 1650, 13697.

Product data

One tub contain: 200 ready to use impregnated wipes.
1 g Active ingredient solution contains: 0,25 g Ethanol (94 %), 0,35 g Propan-1-ol.

Chemical / physical data (Active ingredient solution):

Appearance:	clear, colourless liquid
Density (20 °C):	ca. 0,90 g/cm ³
Flash point (DIN 51755):	27 °C

Special notes

Use disinfectants safely. Always read the label and product information before use.

Wearing of protective gloves (e.g. made of butyl rubber) is recommended.

perform[®] classic wipes EP

Forms of delivery/packaging units

Container sizes	Packaging units	Art. No.
200 jumbo wipes dispenser tub	10 tubs	109196

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinions and information

Please visit our website for an overview of all available literature / reports on perform[®] classic wipes EP:
www.schuelke.com

For further information please contact your local representative or local offices.



Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).



Schülke & Mayr GmbH
22840 Norderstedt, Germany
Phone +49 (0) 40 - 521 00 - 0
Fax +49 (0) 40 - 521 00 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1AT
Phone +44 (0) 1142 - 54 35 - 00
Fax +44 (0) 1142 - 54 35 - 01
mail.uk@schuelke.com

Schülke & Mayr Ges.m.b.H
Seidengasse 9
1070 Wien, Austria
Phone +43 (0) 1 - 523 25 01 - 0
Fax +43 (0) 1 - 523 25 01 - 60
office@schuelke.at