

schülke -†



Alcohol-based active solution for disinfecting surfaces

perform® classic alcohol EP

At a glance:

- Fast, effective and residue free
- Ready to use
- Colour and perfume free
- Efficacy tested to European norms

Applications

For disinfection of surfaces in areas where fast microbiological effectiveness is required.

Instructions for use

perform® classic alcohol EP is ready to use and is applied undiluted. Completely moisten surfaces, equipment or tools and allow the solution to work. Remove any excess preparation with a disposable cloth. Do not use to treat surfaces that are particularly sensitive to alcohol, such as acrylic glass.

Microbiological effectiveness

perform® classic alcohol EP is:
bactericidal, fungicidal, virucidal (BVDV, Vaccinia).

Contact times

bactericidal	1 Min.
fungicidal	1 Min.
virucidal (BVDV, Vaccinia)	1 Min.



Tested according to EN 1276, 1650, 13697.

Product data

Composition: 100 g contain: 25 g Ethanol (94 %), 35 g Propan-1-ol.

Chemical / physical data:

Appearance:	clear, colourless liquid
Density at 20 °C:	0,89 g/cm ³
Flash point (DIN 51755):	27 °C

Labelling according to EC Directives

- Xi: Irritant.
R10: Flammable.
R41: Risk of serious damage to eyes.
R67: Vapours may cause drowsiness and dizziness.
S23: Do not breathe vapour.
S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
S35: This material and its container must be disposed of in a safe way.
S39: Wear eye/face protection.
S51: Use only in well ventilated areas.

Special notes

Use disinfectants safely. Always read the label and product information before use.

Wearing of protective gloves (e.g. made of butyl rubber) is recommended.

perform[®] classic alcohol EP

Forms of delivery/packaging units

Container sizes	Packaging units	Art. No.
1 L bottle	10 x 1 L	113908
10 L canister	1 x 10 L	113909
200 L drum	1 x 200 L	113910

Accessories/dispensing aids

	Art. Nr.
1 L S&M spray pump	180112
10 L canister outlet tap	135501
10 L canister spanner	135810
Decanting funnel	117901
Compressed-air nozzle	120001

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinions and information

Please visit our website for an overview of all available literature / reports on perform[®] classic alcohol EP:
www.schuelke.com

For further information please contact your local representative or local offices.



Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).



Schülke & Mayr GmbH
22840 Norderstedt, Germany
Phone +49 (0) 40 - 521 00 - 0
Fax +49 (0) 40 - 521 00 - 318
www.schuelke.com
mail@schuelke.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1AT
Phone +44 (0) 1142 - 54 35 - 00
Fax +44 (0) 1142 - 54 35 - 01
mail.uk@schuelke.com

Schülke & Mayr Ges.m.b.H
Seidengasse 9
1070 Wien, Austria
Phone +43 (0) 1 - 523 25 01 - 0
Fax +43 (0) 1 - 523 25 01 - 60
office@schuelke.at