## schülke wipes safe & easy

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
bagless system

### Step 1

We recommend that you disinfect your hands and the work surface and wear an overall, protective goggles and gloves\*. This will protect you and the schülke wipes.

\* as required by paragraph 4.1.3 of TRBA 250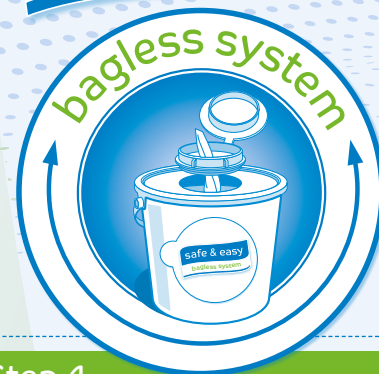

### Step 2

Open the bag by peeling off the self adhesive sealing label.

Make sure you work hygienically. Do not let your gloves become contaminated during the procedure.


### Step 3

Stick the sealing label on the appropriate side of the dispenser box.

### Step 4

Find the innermost wipe in the roll and pull it out slightly. Pour 2 litres of ready-to-use product\*\* or 2.5 litres of diluted concentrate\*\*\* solution once slowly in a spiral shape over the roll of wipes in the bucket.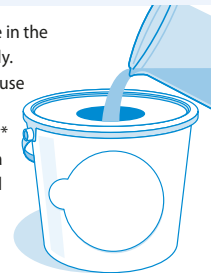

### Step 5

Pull the innermost wipe in the roll through the star-shaped opening.

### Step 6

Screw the lid onto the bag.

### Step 7

Throw away the first wipe and thoroughly close the protective cap.

### Step 8


Write all the necessary information on the sealing label.

### Step 9

The wipes take 30 minutes to become fully saturated.

Wait 30 minutes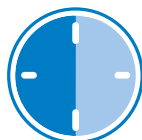

### Step 10

The schülke wipes safe & easy are now ready to use.

Please dispose the bucket with any remaining product and wipes in the correct way after the expiry date of 28 days has been reached. Not intended to be reprocessed.

\*\* acryl® des, antifact® N liquid, mikrozyd® AF liquid, mikrozyd® sensitive liquid, pursept® A Xpress, terralin® liquid, buraton® rapid, perform® advanced alcohol EL, perform® classic alcohol EP

\*\*\* antifact® AF (N), antifact® extra, antifact® FF, Desinfektions-Reiniger AF, pursept® AF, terralin® protect, perform® classic concentrate Q-Plus, TPH protect