



Neutraliser based on phosphoric acid for the automated reprocessing of medical devices and laboratory accessories.

thermosept® NKP

Our Plus

- tenside-free
- foam-free
- broad spectrum of applications
- for neutralisation and acidic cleaning

Application areas

thermosept® NKP is used as neutralizer during the automated reprocessing of surgical instruments after alkaline cleaning in order to avoid propagation of alkaline residues. Furthermore, it can also be used for acidic pre-cleaning, as a basic cleaner for instruments and machine interiors and as a cleaner in a manual immersion bath.

Before applying thermosept® NKP, make sure that the items to be cleaned and reprocessed are acid-resistant.

thermosept® NKP is suitable for use in all common washer/disinfectors and multichamber washer/disinfectors.

Instructions for use

Dosage:

after alkaline cleaning: 0.1 - 0.2 % (1 - 2 ml/l). Dosing is performed by machine-integrated dosing pumps.

Basic cleaning of instruments and machine interiors:

- as an acidic pre-cleaner: 0.2 - 2 % (2 - 20 ml/l)
- basic cleaning of instruments and machine interiors: approx. 4 %, (40 ml/l)
- as a cleaner in a manual immersion bath: approx. 4 %, (40 ml/l)

thermosept® NKP is not suitable for coloured metals, chromium plated parts and anodized aluminium. Please make sure that the items to be cleaned, the machine and the drains are acid resistant. Using demineralised water for final rinsing is recommended and improves the rinsing result. thermosept® NKP can be used in washer/disinfectors and multi-chamber

washer/disinfectors. Please note recommendations of machine and instrument manufacturers.



Product data

Composition:
Contains orthophosphoric acid.

Chemical-physical data

Color	colourless
Density	ca. 1,43 g/cm ³ / 20 °C
Flash point	Not applicable
Form	liquid
pH	ca. 1,2 / 20 °C
Viscosity, dynamic	No data available

Information for order

Item	Delivery form	Item no.
thermosept NKP 6 kg KA	1/Canister	on request
thermosept NKP 20 l KA	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

Application aids	Item no.
Key for 5/10 lt can (Draak)	135801

Related Products

- thermosept® alka clean forte
- thermosept® RKF

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: <http://www.schuelke.com/>.
For individual questions: Customer Sales Service Phone: +49 40 52100-666 E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com