



Alcohol-based, rapid disinfection of non-invasive medical devices and surfaces.

mikrozid[®] AF wipes

Our Plus

- broad efficacy within very short time
- residue-free
- ready to use

Application areas

The product is suitable for disinfecting medical devices and other surfaces (PT2). Our classic product among the rapid disinfectant products is suitable for disinfecting near-patient surfaces with an increased risk of infection that require short exposure times.

Instructions for use

Thoroughly wipe the surface with the cloth and allow the active ingredient solution taking effect. Ensure complete wetting and maintain the surface moist during the entire exposure time. Ensure that all visible dirt is removed before disinfection. Treat only small areas. The wiping area depends on the room temperature and the structure of the surfaces to be disinfected. Pay attention to hygienic aspects when opening and threading the first cloth. Make sure to close the lid after use.

Refilling the box

Disinfect your hands first and put on a fresh pair of protective gloves and a gown. Ensure that the gloves are not contaminated by other activities during the filling process. Transfer the sticker with the expiry date to the dispenser box. Unscrew the lid, place the refill bag completely in the dispenser box (if necessary, shape it to fit beforehand) and cut off the edge of the bag. Pull the pre-plucked cloth from the middle of the roll, thread it from below into the guide slot provided in the cover and pull it through. Disinfect the lid and screw it on. For high-risk areas, schülke recommends not using refill bags.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal EN13727, EN16615 - dirty conditions	ready-to-use	1 min.
tuberculocidal EN14348 - dirty conditions	ready-to-use	1 min.
yeastcidal EN13624, EN16615 - dirty conditions	ready-to-use	1 min.
virucidal against enveloped viruses EN14476 - dirty conditions	ready-to-use	30 sec.
limited spectrum virucidal activity EN14476 - dirty conditions	ready-to-use	30 sec.
Polyoma SV40 in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline	ready-to-use	5 min.
Rotavirus in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline	ready-to-use	30 sec.

Certificates

- ÖGHMP certificate
- VAH certificate
- IHO listed



mikrozid[®] AF wipes

Product data

Composition:

100 g solution contains the following active substances:
24,75 g Ethanol (94 %), 35 g Propan-1-ol

Chemical-physical data

Color	colourless
Density	approx. 0,89 g/cm ³ / 20 °C / of the active solution
Flash point	27 °C / Method : DIN 51755 Part 1 / of the active solution
Form	Aqueous, alcohol containing solution on non-woven
pH	Not applicable
Viscosity, dynamic	No data available

Special advice

Use disinfectants safely. Always read the label and product information before use.

The product has good material compatibility with metals and plastics (except acrylic glass and alcohol-sensitive paints). Surfaces that are particularly sensitive to alcohol, such as acrylic glass, must not be treated. When using alcoholic disinfectants, please observe the fire and explosion protection in accordance with the trade association regulation "Disinfection Work in Health Care". Keep away from heat and sources of ignition. Wearing protective gloves is advised. Further information is available on request. If handled properly, schülke guarantees a shelf life of 3 months after the package has been opened. **For professional use only.** Any serious incidents related to the product are to be reported to the manufacturer and competent authority. Not for final disinfection of semicritical and critical medical devices!

Information for order

Item	Delivery form	Item no.
mikrozid [®] AF wipes Refill (150)	20/Carton	on request
mikrozid [®] AF wipes premium softpack (50)	12/Carton	on request
mikrozid [®] AF wipes Jumbo Refill (220)	12/Carton	on request
mikrozid [®] AF wipes tub (150)	10/Carton	on request
mikrozid [®] AF wipes Jumbo tub (220)	10/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

Application aids	Item no.
wall bracket Jumbo tub	134421
Wall mounting for the dispenser (small)	134417

Related Products

- mikrozid[®] PAA wipes
- mikrozid[®] sensitive wipes
- mikrozid[®] universal wipes premium

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com.

For individual questions:

Application Department

Phone: +49 40 52100-666

E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com