



Cleaner for automated reprocessing
of dental instruments and accessories

thermodent® alka clean

Our Plus

- to minimise the risk of iatrogenic transmission of CJD/ vCJD (in accordance with RKI notification)
- protects the materials very good cleansing power
- good wetting due to the addition of surfactants

Application areas

thermodent® alka clean is suitable for the efficient cleaning and reprocessing of dental instruments, MIC instruments, containers, trays, handpieces and angle pieces. thermodent® alka clean is suitable for use in all common washers/ disinfectors (WD).

Instructions for use

Use concentration: thermodent® alka clean is used in the following proportions: Standard recommendation: 0.5 % (5 ml/l), depending on level of soiling: 0.4 - 1 % (4 - 10 ml/l) possible. Cleaning time approx. 5 - 10 min. at 40 - 65 °C (standard temp. 55 °C)

Application tips: the rinsing results could be improved by using demineralized water and adding thermodent® clear to the final rinse. Following the cleaning step, it is advisable to neutralize with thermodent® neutralizer to prevent the entrapment of alkaline residues.

The product is proportioned by the machine's own dosing pumps.



Product data

Concentration:

Labelling according to regulation (EC) No. 648/2004:

< 5 % nonionic surfactants, < 5 % anionic surfactants, < 5 % NTA and its salts, < 5 % polycarboxylates, enzymes, < 5 % NTA (nitrilotriacetic acid) and salts thereof.

Other ingredients: solubilizer, corrosion inhibitor.

Chemical-physical data

Color	nearly colourless
Density	ca. 1,08 g/cm ³ / 20 °C / 1.013 hPa
Flash point	> 70 °C / Method : DIN 51755 Part 1
Form	liquid
pH	12 - 12,8 / 20 °C
Viscosity, dynamic	ca. 3 mPa*s / Method : ISO 3219

Special advice

Please check material compatibility of alkaline sensitive materials (e.g. anodized aluminium) before use. Do not mix product concentrate with other chemicals.

Information for order

Item	Delivery form	Item no.
thermodent [®] alka clean 5 l	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

Application aids	Item no.
Can key for 5 + 10 l	135810

Related Products

- thermodent[®] clear
- thermodent[®] neutralizer

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com. For individual questions: Customer Sales Service Phone: +49 40 52100-666 E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com