



70 % (v/v) IPA ready to use solution for disinfecting surfaces.

perform® classic alcohol IPA

Our Plus

- fast, effective and residue free
- ready to use
- colour and perfume free
- efficacy tested according to European norms (EN)

Application areas

For disinfection of surfaces in areas where fast microbiological effectiveness is required.

Instructions for use

Apply perform® classic alcohol IPA undiluted to objects and surfaces and allow to take effect. Ensure complete wetting. The product dries off quickly. Maximum application rate 50 ml/sq.m. Do not treat surfaces that are particularly sensitive to alcohol such as acrylic glass.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>EN1276, EN13697</i>	ready-to-use	1 min.
yeasticidal <i>EN1650, EN13697</i>	ready-to-use	1 min.
virucidal against enveloped viruses <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i>	ready-to-use	30 sec.



perform[®] classic alcohol IPA

Product data

Composition:

100 g solution contains the following active substances: 63,1 g Propan-2-ol (equates 70 % (v/v)).

Chemical-physical data

Color	colourless
Density	approx. 0,88 g/cm ³ / 20 °C
Flash point	22 °C / Method : DIN 51755 Part 1
Form	liquid
pH	Not applicable
Viscosity, dynamic	> 2,5 mPa*s / 20 °C / Raw material

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com.

For individual questions:

Application Department

Phone: +49 40 52100-666

E-Mail: info@schuelke.com

Special advice

Use disinfectants safely. Always read the label and product information before use.

EU-0023656-0001 1-1

Information for order

Item	Delivery form	Item no.
perform [®] classic alcohol IPA 1 l bottle	10/Carton	on request
perform cl. alc IPA EN/IT/E 1 l FL	10/Carton	on request
perform [®] classic alcohol IPA 10 l canister	1/Canister	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

Application aids	Item no.
Can key for 5 + 10 l	135810
Compressed-air nozzle	120001
Decanter funnel	117901
schülke tap for 5 l / 10 l can	135501
Spray head for 1l bottle	180212

Related Products

- perform[®] classic mix dry wipes

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com