

dosage table gigasept® pearls

schülke →

measuring beaker: 100 g granulate \triangleq 150 ml

1 heaped measuring spoon \triangleq 40 g granulate \triangleq 60 ml

litre solution	concentration of the working solution	
	1 %	2 %
1 litre	15 ml (\triangleq 10 g)	30 ml (\triangleq 20 g)
2 litre	30 ml (\triangleq 20 g)	60 ml (\triangleq 40 g)
4 litre	60 ml (\triangleq 40 g)	120 ml (\triangleq 80 g)
8 litre	120 ml (\triangleq 80 g)	240 ml (\triangleq 160 g)
10 litre	150 ml (\triangleq 100 g)	300 ml (\triangleq 200 g)
20 litre	300 ml (\triangleq 200 g)	600 ml (\triangleq 400 g)
30 litre	450 ml (\triangleq 300 g)	900 ml (\triangleq 600 g)

Instructions for use:

First add the water. Sprinkle in the corresponding amount of granulate for the volume of water added using the measuring beaker/spoon. Stir several times for the first 15 minutes. After this activation time, the working solution is ready for use. According to recommendations by the German Robert Koch Institute, replace working solutions every working day and if contamination is clearly visible.