



pH-neutral, enzyme based cleaner for automated reprocessing of flexible endoscopes and medical instruments

thermosept® ER

Our Plus

- suitable for flexible endoscopes and medical instruments
- pH-neutral with excellent material compatibility
- contains enzymes to support the cleaning of protein-containing soiling

Application areas

thermosept® ER has been specifically developed for the cleaning of flexible endoscopes in washer-disinfectors (WD). The suitability of thermosept® ER was tested in extensive cleaning efficacy and material compatibility tests. thermosept® ER is suitable for use in WD for endoscopes of all leading manufacturers, including BHT, Olympus and Wassenburg. thermosept® ER is compatible with the disinfectant components thermosept® ED and thermosept® PAA.

Instructions for use

Dosage:

Standard concentration:

0.5 % (5 ml/l); 5 min.; 45 °C

Dosage and cleaning time depending on the level of soiling:

0.4 % - 0.6 % (4 - 6 ml/l); 5 - 10 min.; 35 °C - 55 °C.

Flexible endoscopes and additional endoscopic equipment can be disinfected by using thermosept® PAA after cleaning step, both products are compatible. The use of demineralised water is recommended.

Note: It is not possible to combine thermosept® ER with thermosept® PAA in short programmes, in which the disinfectant is pumped into the cleaning agent.

The dosage and application temperatures of the product are to be adjusted via the machines.



Product data

Composition:

Labelling according to Regulation (EC) No. 648/2004:

5 - 15 % non-ionic surfactants, enzymes. Contains Alkylpolyethylen-glycol-polybutylen-glycolether. Other ingredients: solubilising agents, corrosion inhibitors.

Special advice

Always read the label and product information before use.

For professional use only. Any serious incidents related to the product are to be reported to the manufacturer and competent authority. Please note recommendations of machine and instrument manufacturers. Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container.

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit www.schuelke.com

Information for order

Item	Delivery form	Item no.
thermosept ER 2 I FL	6/Carton	70000333
thermosept ER 2 I FL	6/Carton	70003421
thermosept ER 5 I KA	1/Canister	189901
thermosept ER 5 I KA	1/Canister	70003422

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or its use please call 0114 2543500 or E-mail mail.uk@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke UK
Schülke & Mayr UK Ltd
Cygnet House, Meadowhall
Sheffield, S9 1AT
Phone +44-114-254 35 00
Fax +44-114-254 35 01
mail.uk@schuelke.com

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com