



Unique alcohol formulation for virucidal protection in the cleaning and disinfection of hard surfaces.

mikroqid[®] wiperes

Our Plus

- blend of two alcohols for a very high performance against viruses within one minute contact time (EN14476)
- cleaning and disinfection, two complete actions in one product
- added surfactants for boosted cleaning performance
- effective against bacteria (inc. TB), yeast and viruses, including enveloped viruses
- quick drying and residue free
- effective against bacteria within one minute contact time (EN16615 4-field-test)
- jumbo: 220 wipes (20 x 20 cm) per tub/refill pack
- ready to use

Application areas

Alcohol-based rapid disinfection of non-invasive medical devices / surfaces in all areas with an increased risk of infection and where short contact times are required, e.g.:

- Patient treatment units and nearby surfaces
- Operating tables and nearby work surfaces
- Surfaces of medical equipment

The area a wipe will cover depends on the room temperature and type of surface to be disinfected. More information available on request.

Instructions for use

Wipe surfaces thoroughly with the impregnated wipe, ensure complete wetting and allow to take effect. Make sure that all visible soiling is removed before disinfection and the lid is tightly closed after use. If necessary, use several wipes for large areas.

Do not treat surfaces that are sensitive to alcohol, e.g. acrylic glass. Not for final disinfection of semi-critical and critical medical devices.

Refilling

First disinfect hands and wear new protective gloves. During the refilling process, please ensure that the gloves are not contaminated by other activities. On every refill, subject the lid to disinfectant cleaning with an alcohol based surface disinfectant.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
MRSA	ready-to-use	1 min.
tuberculocidal	ready-to-use	1 min.
yeastcidal <i>in accordance with VAH Guidelines with short contact times</i>	ready-to-use	1 min.
virucidal against enveloped viruses <i>- dirty conditions</i>	ready-to-use	1 min.
Vaccinia virus	ready-to-use	30 sec.
Norovirus <i>EN14476</i>	ready-to-use	1 min.
Rotavirus	ready-to-use	30 sec.

CE 0297



Product data

Composition:

One standard tub / refill pack contains contains 220 ready to use wipes

100 g solution contains the following active substances:

0.25 g Ethanol (94 %), 0.35 g 1 Propanol

Special advice

Use biocides safely. Always read the label and product information before use.

For all of the related safety information please refer to the Safety Data Sheet. To view the Safety Data Sheet please visit www.schuelke.com

Wearing gloves (e.g. of nitrile rubber) is recommended. Keep container tightly closed. Protect from frost, heat and sunlight. Store at room temperature in the original container. Only recycle completely emptied packaging. Leftover product can be disposed of with household waste. Do not use near ignition sources. Do not exceed usage of more than 50ml per m².

Avoid contact with the eyes. Store out of the reach of children. Do not use on alcohol sensitive surfaces (e.g. leather, PVC or acrylic).

The wiping range depends on the room temperature and the structure of the surface to be disinfected.

schülke guarantees, if used in the correct way, a shelf life of 3 months after the package has been opened.

Order reference codes:

Item No. 70002980 GTIN: 4032651980321

Item No. 70002981 GTIN: 4032651980345

Expert opinion and information

Please visit our website for an overview of all available literature/reports on all schulke products online at <http://www.schuelke.com/>

For any queries you might have about the product or using it please call 0114 2543500 or E-mail mail.uk@schuelke.com

* The surface coverage of one wipe depends on the surrounding conditions and the structure of the wiped surface.

Information for order

Item	Delivery form	Item no.
mikrozid wipes Ref GB 150 ST BT	20/Carton	70002065
mikrozid wipes Jumbo Ref -GB- 220 ST BT	12/Carton	70002981
mikrozid wipes Jumbo Ref -GB- 200 ST BT	12/Carton	70001450
mikrozid wipes Jumbo -GB- 220 ST DS	10/Carton	70002980
mikrozid wipes -GB- 150 ST DS	10/Carton	109216
mikrozid wipes Jumbo -GB- 200 ST DS	10/Carton	109214

Related Products

- mikrozid[®] AF wipes premium
- mikrozid[®] alcohol free wipes
- mikrozid[®] PAA wipes
- mikrozid[®] universal wipes

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke UK
Schülke & Mayr UK Ltd
Cygnet House, Meadowhall
Sheffield, S9 1AT
Phone +44-114-254 35 00
Fax +44-114-254 35 01
mail.uk@schuelke.com

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com