



gigasept® AF forte

gigasept® AF forte is an aldehyde-free preparation for cleaning and disinfection of rigid and flexible endoscopes as well as surgical instruments.

Our Plus

aldehyde-free
suitable for flexible and rigid endoscopes
shortened action time in ultrasonic baths
patented substance combination

Downloads

 [Product Information Sheet, PDF \[400.6 KB\]](#)

[To all downloads](#)

Source of supply



Professional

Fields of Application



Cleaning and disinfection of flexible and rigid endoscopes, surgical instruments, anaesthesia equipment and breathing masks.

Instructions for use



gigasept® AF forte is a concentrated product and is diluted with cold water to the desired working concentration.

Reprocess flexible endoscopes with an application concentration of no more than 2%!

Dosage:

Depending on the degree of effectiveness 0.5 % - 5 %.

Application example:

1 litre of a 5 % working solution corresponds to 950 ml of water and 50 ml of gigasept® AF forte.

Immerse instruments to be reprocessed into the gigasept® AF forte solution immediately after use. Make sure that all surfaces are completely wetted and allow them to soak in the solution. Submerge instruments with a hollow body such that the air can fully escape from the lumen. After use, thoroughly rinse/flush the instruments with water that has at least drinking water quality, or better yet with sterile distilled or demineralised water, to remove any residues of the working solution.

Do not exceed indicated contact times and concentrations. Please note the treatment recommendations provided by the instrument manufacturers. National regulations may require that cleaning and disinfection are carried out in two separate process steps.

Not suitable for the final disinfection of semi-critical and critical medical devices!

Standing time:

A standing time of 7 days applies if the working solution is only used for the disinfection of previously cleaned instruments. If the working solution is used for cleaning, national regulations may limit the timespan during which the solution may be used.

Certificates / Norms



Product data



Composition:

100 g agent solution contains the following active ingredients:

10 g Dimethyldioctylammonium chloride, 15 g Phenoxypropanols, 15.6 g Alkylguanidine acetate, 9.5 g Laurylpropylene diamine.

Special advice



Always read the label and product information before use.

For professional use only.

Any serious incidents related to the product are to be reported to the manufacturer and competent authority. Do not use the product after the shelf-life has expired. Keep container tightly closed. Protect from frost, heat and sunlight. Store in the original

container.

When using 5% stock solutions, accidental contact with the eyes can result in irritation of the mucosa.

Due to the choice of highly effective ingredients effects of turbidity (opalescence) might appear at the recommended use concentrations. These effects are product specific and do not limit the efficacy of the product.

Besides manual reprocessing gigasept® AF forte is also suitable for semi-automated processes at room temperature.

Interactions:

As a general rule, do not mix gigasept® AF forte with other products. If gigasept® AF forte comes into contact with aldehyde-containing working solutions, precipitation and irreversible discolouration may occur.

Information for order



Item	Delivery form	Item
gigasept AF forte -INT- 5 l KA	1 / canister(s)	on request
gigasept AF forte -INT- 2 l FL	5 / carton(s)	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids



Item	Delivery form
schülke dosing feeder 5 l / 10 l (20ml / stroke)	
tap for 5 l / 10 l can	
Can key for 5 + 10 l	
doser for 2 lt bottles	

Related products



[gigasept® pearls](#)

Environmental information



schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Downloads

-  Safety data sheet, Switzerland | Italian, PDF [562.3 KB]
-  Safety data sheet, Belgium | French, PDF [566.4 KB]
-  Safety data sheet, Germany | German, PDF [524.7 KB]
-  Safety data sheet, Belgium | German, PDF [515.3 KB]
-  Safety data sheet, Czech Republic | Czech, PDF [576.6 KB]
-  Safety data sheet, United Kingdom | English, PDF [423 KB]
-  Safety data sheet, Austria | German, PDF [517.7 KB]
-  Safety data sheet, France | French, PDF [569.5 KB]
-  Safety data sheet, Belgium | Dutch, PDF [514 KB]
-  Safety data sheet, Netherlands | Dutch, PDF [516.1 KB]
-  Safety data sheet, Switzerland | French, PDF [575.3 KB]
-  Safety data sheet, Switzerland | German, PDF [528.1 KB]
-  Safety data sheet, Slovakia | Slovakian, PDF [577.3 KB]
-  Safety data sheet, Poland | Polish, PDF [590.9 KB]

More information

-  Product Information Sheet, PDF [400.6 KB]

Assets

gigasept_AF_forte_2l_910940-prod_master_pic-IP-World_en-SMDE.png	46282f2d-8d2a-4d25-a621-1d2f4c34912e
Z40000283gigasept_af_forte_910940-bmsds-IP-CH_it-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-CH_it-SMDE.pdf
MC1905-mat_comp_report-IP-DE_de-SMDE.pdf	f81ee16f-7425-4210-a965-106454fb15d5
MC2228-list_disinfect-IP-WORLD_en-SMDE.pdf	9d3591a4-7d4b-4c58-b54c-12365cf42e93
MC1507-instruc_prep-IP-DE_de-SMDE.pdf	eca78895-8641-4da6-9e2b-69e5101ba498
MC1461-list_disinfect-IP-WORLD_en-SMDE.pdf	326adb92-9c0f-4d42-b399-e34cc71cf571
MC1119-list_disinfect-IP-DE_de-SMDE.pdf	d472e0f7-3cd0-43ee-8a75-5d2a7b62efec
MC2117-list_disinfect-IP-World_en-SMDE.pdf	e6d7321c-042f-42a0-b718-f8c8e535e543
MC2164-list_disinfect-IP-WORLD_en-SMDE.pdf	1a85c907-a080-4b5d-a461-e1538731eab3
Z40000283gigasept_af_forte_910940-bmsds-IP-BE_fr-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-BE_fr-SMDE.pdf
Z40000283gigasept_af_forte_910940-bmsds-IP-DE_de-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-DE_de-SMDE.pdf
MC0359-list_disinfect-IP-DE_de-SMDE.pdf	6ac47f03-05c7-4079-bc69-8e42fc8edf75
MC0104-list_disinfect-IP-DE_de-SMDE.pdf	7fcbd0f8-a315-4a95-8ebd-17e3ef091e24
MC0585-list_disinfect-IP-DE_de-SMDE.pdf	b079a2d8-035a-43e2-873a-08cad6b0d6a4
Z40000283gigasept_af_forte_910940-bmsds-IP-BE_de-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-BE_de-SMDE.pdf
MC0298-mat_comp_report-IP-DE_de-SMDE.pdf	a1ead4fa-6be5-4bcb-8aaf-773dce80097d
MC0300-mat_comp_report-IP-DE_de-SMDE.pdf	1434d8ca-5d2d-4c18-9dc3-7e90c8b64982

MC0584-list_disinfect-IP-DE_de-SMDE.pdf	b4580fd0-5145-4584-80e0-8972ecd79b91
MC0558-list_disinfect-IP-DE_de-SMDE.pdf	a045d937-967c-423b-8200-eaaa6f6872df
MC0302-mat_comp_report-IP-DE_de-SMDE.pdf	b6a2b8f8-7ac9-42b0-afcc-216856a3a717
MC1906-mat_comp_report-IP-WORLD_en-SMDE.pdf	c31bfd3a-6007-4e5d-8dd0-a1b14bda7b2a
MC0297-mat_comp_report-IP-WORLD_en-SMDE.pdf	c0b603bb-28e2-4633-8e03-629b2dbfb5e0
MC1450-list_disinfect-IP-DE_de-SMDE.pdf	d5d967a8-a898-42d4-803b-eb91f07e344c
MC2328-list_disinfect-IP-DE_de-SMDE.pdf	2ca489d7-b393-4e93-a553-3ae99e7d603a
Z40000283gigasept_af_forte_910940-bmsds-IP-CZ_cs-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-CZ_cs-SMDE.pdf
MC0587-list_disinfect-IP-DE_de-SMDE.pdf	8941bb5c-eea8-481e-8e7a-4939be584fb3
Z40000283gigasept_af_forte_910940-bmsds-IP-GB_en-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-GB_en-SMDE.pdf
MC1204-list_disinfect-IP-DE_de-SMDE.pdf	125eac23-3997-465b-9268-cd92a825410e
MC2279-list_disinfect-IP-DE_de-SMDE.pdf	a5052a07-1df1-4b13-9bf3-d175ef1edfd6
MC0299-mat_comp_report-IP-WORLD_en-SMDE.pdf	df8a3a4f-aa68-4691-977a-1a3e6776eb4c
MC0528-list_disinfect-IP-WORLD_en-SMDE.pdf	edbf7eaa-d00d-422e-b4f0-f930aa28acba
Z40000283gigasept_af_forte_910940-bmsds-IP-AT_de-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-AT_de-SMDE.pdf
MC0557-list_disinfect-IP-WORLD_en-SMDE.pdf	ba4b2ef9-a749-4e2e-ad62-826f4d1ee0ab
Z40000283gigasept_af_forte_910940-bmsds-IP-FR_fr-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-FR_fr-SMDE.pdf
MC1203-list_disinfect-IP-DE_de-SMDE.pdf	a9e69622-f6f5-4628-8bf4-6d0c98a17f69
Z40000283gigasept_af_forte_910940-bmsds-IP-BE_nl-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-BE_nl-SMDE.pdf
Z40000283gigasept_af_forte_910940-bmsds-IP-NL_nl-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-NL_nl-SMDE.pdf
MC0979-list_disinfect-IP-DE_de-SMDE.pdf	7ac012a3-7dcd-459c-a169-84b2f8079b36
MC2109-mat_comp_report-IP-WORLD_en-SMDE.pdf	982326a1-41f6-4dad-b33d-f84a22de9a0f
MC0590-list_disinfect-IP-DE_de-SMDE.pdf	318071f6-cf7e-44eb-96eb-6906e99a730e
MC1492-list_disinfect-IP-DE_de-SMDE.pdf	2450db16-5a2f-4df3-a7b0-998fbe972889
MC1123-ifu-IP-DE_de-SMDE.pdf	c0ea7926-517e-41a9-a29b-092d383ef702
Z40000283gigasept_af_forte_910940-bmsds-IP-CH_fr-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-CH_fr-SMDE.pdf
MC1866-mat_comp_report-IP-WORLD_en-SMDE.pdf	1f87d041-f76b-4175-a2cf-6b25c2ac3b2e
MC0583-ifu-IP-WORLD_en-SMDE.pdf	eb5cad4e-db52-4081-b6f1-a6e1b7aa7a85
MC1865-mat_comp_report-IP-DE_de-SMDE.pdf	5b563d23-e159-423f-a2b0-15554230ab59
MC0334-mfr_letter-IP-DE_de-SMDE.pdf	0e67adf1-3024-4f56-a3c9-b6c25db38b02
MC1212-list_disinfect-IP-WORLD_en-SMDE.pdf	820f2ce1-3571-4544-ae89-d2577f3644f4
MC1222-list_disinfect-IP-DE_de-SMDE.pdf	4a1e7a86-6e46-43dd-ae92-aaf21b0127e9
MC0304-mat_comp_report-IP-WORLD_en-SMDE.pdf	ad4cb12f-183d-4545-9d25-61daf728e9b1
MC1172-list_disinfect-IP-DE_de-SMDE.pdf	6614ba92-cc7d-48fb-80bd-5c9dd7e3769c
MC0928-list_disinfect-IP-DE_de-SMDE.pdf	f308df72-9c92-40a3-832e-aec36cbc64d1
MC0305-mfr_letter-IP-DE_de-SMDE.pdf	f5d9553e-e91e-4a4b-a023-288fbbbebed1
MC0527-list_disinfect-IP-WORLD_en-SMDE.pdf	3990dfc7-4269-4b46-a970-f26270c5e7f9
MC0644-list_disinfect-IP-DE_de-SMDE.pdf	745ebdc7-01e3-4127-b4f2-f0fdac9cd8e5
MC0301-mat_comp_report-IP-WORLD_en-SMDE.pdf	517190cc-9c97-408d-b539-6a0c365c66dd
MC0672-list_disinfect-IP-WORLD_en-SMDE.pdf	a212bc19-c17e-4759-9982-c44138aa47a9
MC0547-list_disinfect-IP-DE_de-SMDE.pdf	e2736549-d44d-48bb-b013-1a92e3048b40
MC0103-list_disinfect-IP-WORLD_en-SMDE.pdf	069ad1c3-5df5-45f0-9585-d018ac3024e0
MC0303-mat_comp_report-IP-DE_de-SMDE.pdf	a5d11452-4c1d-4e1b-a111-be04eaa8d036
MC1459-list_disinfect-IP-WORLD_en-SMDE.pdf	b52a5ef8-7525-44bd-ab4f-b6bccca66975
MC0010-list_disinfect-IP-DE_de-SMDE.pdf	f3e162ab-4b91-4b4e-a0a3-5c2d4064ea3f
MC0554-list_disinfect-IP-WORLD_en-SMDE.pdf	b84d4dbc-ec71-49b4-8911-85e6f693abb1
MC1460-list_disinfect-IP-WORLD_en-SMDE.pdf	edd926d2-de78-4ea1-ad46-4d3086a13039
Z40000283gigasept_af_forte_910940-bmsds-IP-CH_de-SMDE.pdf	Z40000283gigasept_af_forte_910940-bmsds-IP-CH_de-SMDE.pdf
MC0559-list_disinfect-IP-DE_de-SMDE.pdf	cee25ae1-0556-4d51-aa7e-91ef4fc1e2ee

Z40000283gigasept_af_forte_910940-bmsds-IP-SK_sk-SMDE.pdf

MC0009-list_disinfect-IP-WORLD_en-SMDE.pdf

MC0589-list_disinfect-IP-DE_de-SMDE.pdf

Z40000283gigasept_af_forte_910940-bmsds-IP-PL_pl-SMDE.pdf

gigasept_AF_forte_910940-PIS_manual-IP-WORLD_en-SMDE.pdf

Z40000283gigasept_af_forte_910940-bmsds-IP-SK_sk-SMDE.pdf

240af384-8445-44c8-891f-a950cdf9a5c7

dd0dae7d-a91f-4b75-9ebc-13663f619509

Z40000283gigasept_af_forte_910940-bmsds-IP-PL_pl-SMDE.pdf

11f22c4d-8eda-416b-ad5b-941180deb7a5