



Ready to use impregnated wipes for disinfecting surfaces.

perform® classic wipes EP

Our Plus

- fast, effective
- economical with 220 jumbo wipes per box
- colour and perfume free
- efficacy tested according to European norms (EN)

Application areas

For disinfection of surfaces in areas where fast microbiological effectiveness is required.

Instructions for use

Tear off wipes downward at an angle. Thoroughly wipe surfaces with perform® classic wipes EP, ensure complete wetting and allow to dry. The product dries off quickly. Close the lid tightly after use. Do not use on surfaces that are particularly sensitive to alcohol, such as acrylic glass.

Microbiological efficacy

Efficacy	Concentration	Contact time
bactericidal <i>EN1276, EN13697</i>	ready-to-use	1 min.
fungicidal <i>EN1650, EN13697</i>	ready-to-use	2 min.
virucidal against enveloped viruses <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i>	ready-to-use	30 sec.



perform[®] classic wipes EP

Product data

One box contains: 220 ready to use impregnated wipes.
100 g active agent solution contains:
25 g Ethanol (94 %), 35 g Propan-1-ol.

Chemical-physical data

Color	colourless
Density	approx. 0,89 g/cm ³ / 20 °C / of the active solution
Flash point	27 °C / Method : DIN 51755 Part 1 / of the active solution
Form	Aqueous, alcohol containing solution on non-woven
pH	approx. 7 / 20 °C / of the active solution
Viscosity, dynamic	No data available

Special advice

Use disinfectants safely. Always read the label and product information before use.

Information for order

Item	Delivery form	Item no.
perform [®] classic wipes EP 220 pieces	10/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com.

For individual questions:

Application Department

Phone: +49 40 52100-666

E-Mail: info@schuelke.com



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 40 - 52100 - 0
Fax +49 40 - 52100 - 318
www.schuelke.com
info@schuelke.com