



Active oxygen-based disinfection of flexible endoscopes

CE 0297

thermosept® PAA additive

Our Plus

- effective against bacteria, mycobacteria, fungi, viruses and spores (incl. Clostridium difficile spores)
- good material compatibility due to a 2-component system
- short contact time

Application area

For the automated chemothermal reprocessing of flexible endoscopes in AERs. thermosept® PAA base/additive is suitable for the use in all common AER systems which are released for the use of peracetic based products.

Bactericidal, tuberculocidal, mycobactericidal, fungicidal, virucidal (in accordance with DVV/ RKI Guideline** 12/2014 and EN 14476, EN 17111), sporicidal (Clostridium difficile spores) at 1.0 % - 37 °C - 5 minutes or 1.2 % - 35 °C - 5 minutes.

Instructions for use

Dosage:

1 % (10 ml/l) - 37 °C or 1.2 % (12 ml/l) - 35 °C

Whenever possible, simultaneously dose thermosept® base and thermosept® additive. If the AER metering pumps are unable to dose the two components simultaneously, add thermosept® PAA base and then thermosept® PAA additive immediately thereafter. Ensure that this order is followed so as to guarantee material compatibility.

The contact time is 5 minutes.

Disinfection components are compatible with the mild alkaline cleaning agent thermosept® EndoCleaner. Approved microbiological compatibility.

Product data

Composition, base component:

100 g of solution contain:

5 g of peracetic acid, hydrogen peroxide and acetic acid

Composition, additive component:

100 g of solution contain: potassium hydroxide, phosphates and corrosion inhibitors

Labelling according to Regulation (EC) No. 648/2004: 5 % phosphates

Special advice

The base and additive components have to be used in combination. Store in original container at room temperature. Keep the container tightly closed. Protect from heat and sunlight. thermosept® PAA is not suitable for instruments made of brass and copper or chrome-plated instruments. According to EN 17664, the instrument manufacturers' instructions should be followed and, if necessary, the compatibility of the materials ensured before the application. This applies in particular when reprocessing older endoscopes. These should be strictly tested for compatibility with peracetic acid.

For professional use only. Read label and use instructions before use.

thermosept® PAA additive

Information for order

Item	Delivery form	Item no.
thermosept PAA additive 5 l KA	2 / carton(s)	on request
thermosept PAA additive 200 l FA	1 / drum(s)	on request
thermosept PAA additive BHT Pin 5 l KA	2 / carton(s)	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Application aids

Item	Item no.
Can key for 5 + 10 l	135810
Decanter funnel	117901
schülke dosing feeder 5 l / 10 l (20ml / stroke)	117101
tap for 5 l / 10 l can	135502

Related products

- thermosept® ER
- thermosept® EndoCleaner
- thermosept® PAA base
- thermosept® X-tra

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product:
www.schuelke.com

For individual questions:

Application Department

Phone: +49 40 52100-666

E-Mail: info@schuelke.com



schülke -+

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Straße 2
22851 Norderstedt | Germany
Phone +49 40 52100582
www.schuelke.com
info@schuelke.com

 Made in Germany



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.