


Instructions for reprocessing of medical devices

Preparation

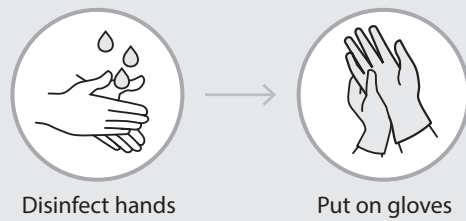

Disinfect your hands and put on suitable protective clothing (PPE).

1. Cleaning



Open the tub of **gigasept® powerSET3 cleaning wipes**, take out a wipe and close the tub again. Unfold the wipe.

Clean the medical device by wiping it thoroughly with firm, rotating motions. Wipe the medical device from proximal to distal until no more soiling is visible and all surfaces have a slight film of moisture. Pay particular attention to joints, edges and recesses. Clean recesses additionally by pressing in a wipe tip 5x in a rotating motion. Avoid re-soiling of the medical device afterwards, e.g. by placing it in a clean holder.

Dispose of the wipe.

Change protective gloves and carry out hand disinfection.

2. Disinfection



Open the tub of **gigasept® powerSET3 disinfection wipes**, take out a wipe and close the tub again. Unfold the wipe.

Wipe the medical device thoroughly from proximal to distal, paying particular attention to joints, edges and recesses. Ensure that all areas are evenly and completely wetted so that all surfaces have a slight film of moisture. Disinfect recesses additionally by pressing in a wipe tip in a rotating motion.

Maintain subsequently a contact time of **5 minutes**. Avoid recontamination during this time, e.g. by placing the medical device in a disinfected holder.

Dispose of the wipe.

3. Rinsing



Open the **gigasept® powerSET3 rinsing wipes** packaging at the marked point. Remove and unfold the wipe.

Wipe the medical device thoroughly until all surfaces are completely wetted and possible residues are removed.


Dispose of the wipe.

Completion



Take off PPE and perform hand disinfection. Complete documentation in the enclosed booklet.

Watch the
video here:

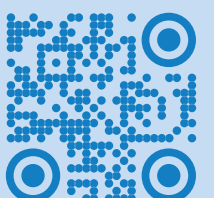

Please read the detailed Instruction Manual before use and follow the application notes.