

gigasept® powerSET3

The new standard in manual reprocessing using wipes

Patient safety is always paramount. Naturally that also includes professional reprocessing of medical devices that are placed on or inserted into patients' bodies. As a specialist in disinfection we offer you a safe solution for manual reprocessing with wipes.

A smart solution.

Exercising due care is a challenge when time is pressing. That's where we come in: The gigasept® powerSET3 is a handy solution you can rely on. The wipes are fully prepared, ready to hand and arranged logically from left to right in the box. The two tubs are firmly attached to the box to prevent mix-ups. The relevant documentation book is always to hand. That keeps the steps to be carried out to the absolute minimum necessary, saving time and preventing errors.

Enhanced safety.

You can rely on the gigasept® powerSET3 when it comes to safety. For patients, you ensure a clean and disinfected medical device within a short time; users benefit from easy handling and seamless documentation of the reprocessing cycle. All components are ready-to-use and thus immediately applicable without preparation or activation. In addition, an antimicrobial coating on the box protects against cross-contamination.



- 1 **gigasept® powerSET3 cleaning wipe**
for cleaning of the medical device
- 2 **gigasept® powerSET3 disinfection wipe**
for high-level disinfection of the medical device
- 3 **gigasept® powerSET3 rinsing wipe**
for rinsing of possible disinfectant residues on the medical device


gigasept® powerSET3 – for reprocessing of the following non-lumened medical devices*:



Transvaginal
ultrasound probes
(TVUS)



Transesophageal
ultrasound probes
(TEE/TOE)

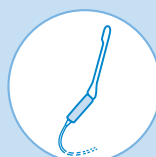

Transrectal
ultrasound probes
(TRUS)



Non-invasive
ultrasound probes



Flexible endoscopes
without channels
(e.g. nasopharyngoscopes)

* Please follow the reprocessing recommendations of the medical device manufacturers. Not suitable for reprocessing of critical medical devices.

The benefits of the gigasept® powerSET3:

- ✓ All-round solution for manual reprocessing including cleaning, disinfection, rinsing and documentation
- ✓ 100% ready-to-use – no preparation or activation needed
- ✓ Comprehensive efficacy (bactericidal, yeasticidal, mycobactericidal, fungicidal, virucidal, sporicidal against *C. difficile*) within 5 minutes
- ✓ Microbiological tests according to latest standards
- ✓ Broad material compatibility for use on different materials
- ✓ Large wipe format (20 x 20 cm) and high-quality non-woven fabric for optimal cleaning and wetting
- ✓ Functional, flexible and space-saving packaging for user-friendly handling
- ✓ Antimicrobial cardboard coating as additional protection against cross-contamination

**gigasept®
powerSET3**

**How to order the
gigasept® powerSET3:**

Carton with 5x gigasept® powerSET3
for 30 applications each

Item no.: 70003677



Would you like detailed information about the
gigasept® powerSET3?

Write to us or give us a call:

✉ info@schuelke.com ☎ +49 40 52100-666

Would you like to learn more right away?
Further information can be found here:

www.schuelke.com/gigaseptpowerset3/int